
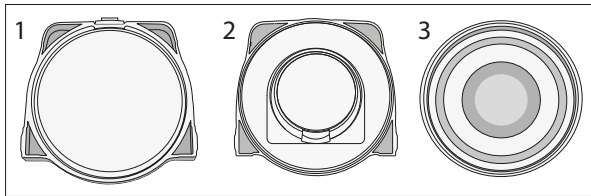
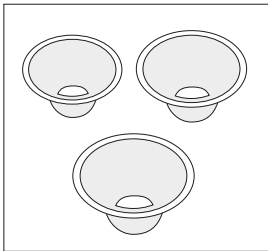
3.2


3.3

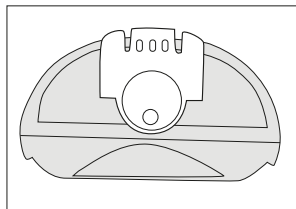
3.4

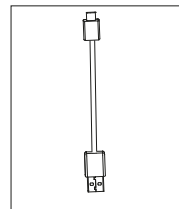
3.5

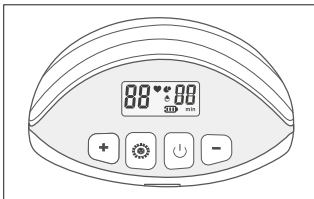
4.1

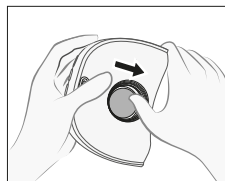5.1

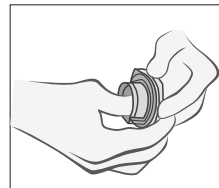
5.2


Wireless Electric In-Bra Breast Pump

Model: 20/100

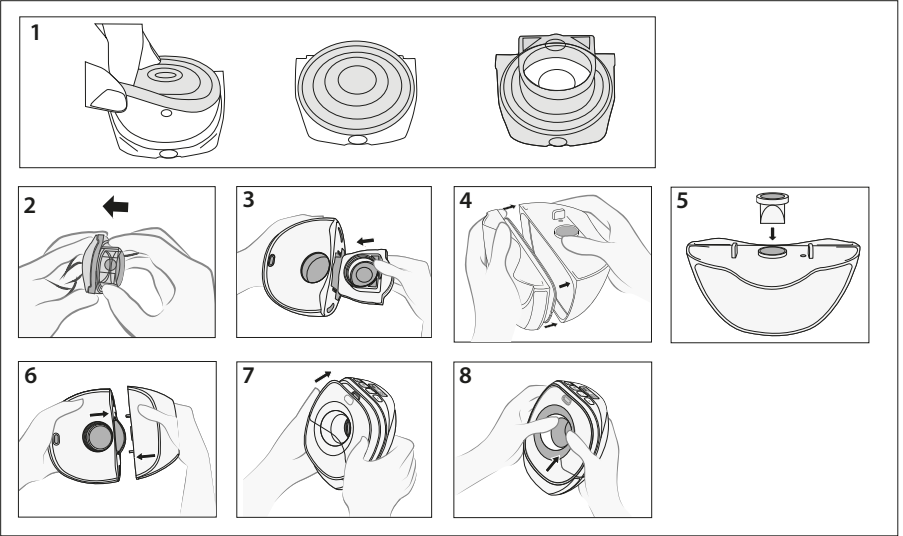
6

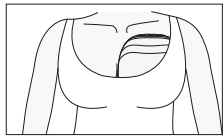
7.1

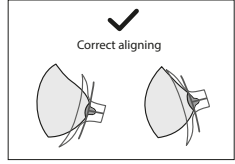7.2

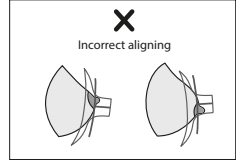7.3

2

3

IMPORTANT SAFETY NOTES

Basic safety precautions should always be followed when using electrical devices, especially when children are around. Please read all instructions before using this product. Please follow the instructions for use, otherwise personal injury or damage to the breast pump may result.

- WARNING**
- Make sure that you are using power adapter compatible with the breast pump's specification.
 - Keep the power adapter away from hot surfaces.
 - When removing the power adapter, please first confirm that the power adapter is separated from the power socket.
 - When placing or storing breast pump, avoid areas where it can easily fall or drop into bathtubs and sinks.
 - The accessories of breast pump can be sterilized by boiling, its motor is not heat resistant, please keep away from heated surfaces or naked flames.
 - Do not use electric breast pump near flammable materials.
 - When the breast pump is powered on, it must be used under supervision.
 - Do not operate if the cable or plug of the electric breast pump is damaged and does not work properly, or, if the breast pump has been dropped or damaged. If you find damage, stop using it immediately and contact service@canpolbabies.com.
 - Do not use electric breast pumps that have been exposed to water or other liquids. If the device falls into the water, do not handle it with your hands, please unplug the power plug immediately.
 - Do not put or drop into water or other liquids.
 - Do not pour water on the breast pump.
 - Do not use the breast pump in the shower or bath.
 - Do not expose the breast pump or the battery to extremely hot environments.
 - Please note that the USB - Type - C port or the battery of the breast pump is at a risk of metal or liquid short circuits.
 - The breast pump cannot be maintained or repaired. Do not repair it yourself. Do not modify the device.

- Use of a damaged breast pump is prohibited. Please replace damaged or worn parts.
- Please use the breast pump only for the purpose described in this manual.
- Do not use the breast pump while sleeping or being too sleepy.
- Do not use the breast pump while driving.
- Please check all appropriate breast pump components before EACH use.
- Breast pumping can cause uterine contractions. Do not pump before giving birth.
- If you become pregnant while breastfeeding or breast pumping, consult a licensed professional doctor before continuing breast pumping.
 - Please do not microwave breast milk or boil it. When breast milk is heated in the microwave, it becomes very hot and is likely to cause severe burns to your baby's mouth. (Microwaving may also change the composition of breast milk.
- Please only use components recommended by Canpol sp. z o.o. with Canpol breast pumps.
 - Close supervision is required if the breast pump or accessories are used within the reach of children.
 - The breast pump is not intended for use by persons with physical, sensory or mental impairments or lack of experience and expertise, including children under the age of 16, provided that they are supervised or instructed to understand how to use the device safely and understand the hazards generated by breast pump.
 - Keep all unused parts out of the reach of children so that they cannot access them to avoid accidental swallowing by children or other accidents. The breast pump and any parts of the breast pump should not be used as toys.
 - Plastic milk collect cup and components will become fragile after freezing and may be broken after dropping.
 - Do not use milk in a damaged milk collect cup or accessory.
 - Contact a breastfeeding specialist in the event of problems or pain. This product is a personal product and there are health risks associated with sharing it with others.
 - Portable or radio frequency communication equipment may interfere with breast pump use.

INSTRUCTIONS FOR USE

The breast pump is an electric breast pump for breastfeeding women to pump and collect breast milk. The breast pump is for personal use.

1. BATTERY MAINTENANCE

- The breast pump contains an built - in (non - replaceable) lithium - ion polymer rechargeable battery.
- CAUTION**
- For a completely empty battery, the breast pump should not be used immediately after being connected to the power adapter, leaving the battery to charge for at least 20 minutes before attempting to turn on the breast pump.
 - In order to keep the breast pump battery in perfect condition, always check that the power adapter used matches the current and voltage required in the product specifications (SV 2A).

- To maintain the lifetime of the batteries .
- Place the breast pump in a cool location.
 - Charge the battery before it runs out of power, this helps to extend the lifetime of the battery.
 - Please fully charge the battery before storing the breast pump.
 - If the breast pump has been stored in a hot location, it may not be used immediately. To restore the normal battery function, cool the breast pump for one hour first. During this period, the breast pump can be charged by plugging the power adapter into a wall socket.

2. CAUTIONS FOR CHARGING

WARNING

Power mode.

- Strictly follow the sequence of steps.
- **Please use 5V2A adapter - not included in the set.**

- Make sure that the power adapter matches the voltage of the local power supply.
- Plug the power adapter USB Type - C cable into the hub. Fig.2.1
- Plug the power adapter into a power outlet. At the moment the hub shouldn't be used. Fig.2.2

3. PRODUCT ACCESSORIES

Fig.3.1

Product:

1. Hub
2. duckbill valve
3. milk collect cup + seal ring
4. flange 24mm
5. Silicone Diaphragm Chamber

Silicone Diaphragm Chamber fig. 3.2:

1. Front cover of chamber
2. Back cover of chamber
3. Silicone diaphragm seal

Silicone breast funnels 3 different sizes 17, 19, & 21mm – fig. 3.3

To know how to choose the correct funnel size, go to:

Stopper for closing milk collect cup. fig.3.4 After expressing the milk, close the container tightly with the white cap. USB - type C cable – fig. 3.5

4. KEY / MODE OPERATION INSTRUCTIONS

Fig. 4.1 Display

♥ Stimulation mode - stimulation of milk flow, fast and shallow sucking. You should change the mode from stimulation mode after 2-3 minutes, unless you notice milk starting to flow earlier.

☘ Expression mode – proper milk expression.

👉 Deep mode – can be used at the end of pumping to further stimulate the breasts/ increase milk production.

The breast pump remembers the last used operating mode, so you can easily use the same settings the next time you express milk. When using two breast pumps at the same time, you can have a different setting on each device.

Key Operation:

🔘 Power on and off: short press the power button to turn on, long press the power button for 1.5s to shut down. Start/ pause: short press the power button to pause, short press the power button to start.

➕ Suction Gear Increase. In working mode, short press „+“ to increase the suction, each press increases the suction by one gear, suction gear P1–P8.

➖ Suction Gear Decrease. In working mode, short press „-“ to decrease the suction, each press decreases the suction by one gear, suction gear P8-P1.

Mode key : short press the mode key to switch modes in working state, and switch in a circular manner.

Low Battery Alert / Auto Shutdown

When the battery is low, the indicator light flashes and automatically powers off after 10 seconds.

Charging Status

The battery indicator flashes when the adapter is connected for charging.

(Fully charged status)

Screen Display Instructions

In working condition, press any key to wake up the screen, the bright screen display time is about 20 seconds.

(Time display status)

Press the suction gear increase or decrease key to switch the corresponding suction gears.

Press the mode key to switch the mode.

Press the pause key to pause the working mode, the breast pumping time is constantly displayed on the screen.

5. CLEANING

Cleaning and disinfection are two different things that must be done separately to care for mother and baby while maintaining the characteristics of the breast pump.

Cleaning - Cleaning of component surfaces by physically removing contaminants.

Disinfection - kills live organisms such as bacteria or viruses that may exist on the surface of the part.

When to clean			
Split before first use	disassemble cleaning disinfect	disassemble cleaning disinfect	disassemble cleaning disinfect
Wash after each use	disassemble cleaning	disassemble cleaning	disassemble cleaning
Once a day	disassemble disinfect	disassemble disinfect	disassemble disinfect

Check all components for damage and replace damaged or worn components. Caution! Care should be taken when cleaning breast pump accessories to avoid damaging them.

As shown on fig.5.1, remove the flange and milk container, and push down the air chamber with your thumb to separate the air chamber from the hub.

As shown on fig.5.2, use your thumb to block the air inlet of the air chamber to make it form a sealed chamber, and the other thumb to press the silicone diaphragm seal, thus opening the front and back covers of air chamber for cleaning.

Cleaning in the sink before first use and after each use.

1. Rinse at parts with clean drinking water (approx. 20°C (68°F)).

2. Wash all parts in plenty of warm soapy water (approx. 30°C (86°F)). Use a detergent that you can purchase, preferably one that is free of artificial fragrances and colours (pH neutral) is recommended.
3. Rinse the parts with clean drinking water (approx. 20°C (68°F)) for 10-15 seconds. Dry after washing.

Disinfection - before first use and once a day.

Fully immerse the milk collect cup, silicone duckbill valve, silicone flange in water, boiling for no more than 5 minutes, dry after disinfection. Dry - wipe to dry with a clean cloth, or place on a clean cloth to dry, place washed parts in a clean storage bag or in a clean environment, do not put wet parts in airtight containers / storage bags. Be sure to dry any residual moisture.

To clean the main unit, the shell can be wiped with a clean, moist (non-dripping) cloth.

6. ASSEMBLING

Assemble the Breast Pump Components fig.6

Disassemble and clean the parts that come in contact with breasts and breast milk immediately after use to avoid breastmilk residue causing bacteria.

1. Place the silicone diaphragm seal on the flat plastic part of the chamber with the convex side up. Then put on the second part of the plastic chamber and tighten it tightly. The convex part of the diaphragm seal should be visible in the opening of the capsule.
2. Make sure, that all the edges of the elements are closely matched to each other.
3. Push the assembled air chamber with the hole side up into the hub, when you hear a click sound, the assembly is complete.
4. Milk container assembly: align milk container front shell with the back shell, press the assembled container around to ensure that the container is completely sealed. There is a special silicone gasket to match elements tight. You will hear a CLICK/RESISTANCE when you assemble the container elements correctly. This will prevent the container from leaking.
5. Place the duckbill valve on the milk outlet hole of the milk container, press down gently to assemble the duckbill valve.

6. Push the assembled milk container into the assembled hub in Figure 3, when you hear the click sound, the milk container assembly is completed.
7. Align the end of the silicone flange with the charging port mark to the charging port of the hub and assemble it to the hub.
8. Both thumbs in downwards, press the center of the silicone flange and push it completely into the hub.

Installation Notes.

CAUTION

- Before touching the breast and breast pump parts, wash your hands thoroughly with water and soap (at least one minute) and dry them with a clean towel or disposable paper towel.
- In addition, milk containers and components may be damaged if they are not properly handled, such as dropped, over-tightened, or knocked over.
- Care should be taken when holding and placing milk containers and components.
- Please only use electric original parts, otherwise there will be air circuit failure that cannot be used.
- Before use, all accessories must be ensured to be completely dry.
- Please perform all steps carefully and assemble the lactation converter accessories correctly, otherwise you may not get a good negative pressure.

7. CORRECTLY WEAR BREAST PUMP

After correctly assembling the breast pump and adjusting the proper position, turn on the breast pump to start working and put it in bra fig. 7.1

SUGGESTION :

- Keep your upper body upright when pumping, lean forward or backward not more than 45°.
- If you lean too much or have a lot of expressed milk (above 100 ml) and you are moving, it may leave the container through the vent hole.
- When you notice milk dripping from under the breast pump, empty the container and continue expressing.

When you have finished pumping, unbutton the nursing bra and slowly take the breast pump out of it. Do not invert the breast pump during the process!

Correct fit to the breast, Fig. 7.2

Uncorrect fit to the breast, Fig. 7.3

Note : Wearing a nursing bra or breast pump for a long time may cause discomfort, so please pay attention to the time you wear and use it.

8. HOW TO POUR & STORE THE MILK ?

When pumping is finished, pull out the hub and duckbill valve, align the milk container with your milk bottle to pour milk. Note: After pumping is finished, please pour the breast milk into your milk bottle for storage as soon as possible. You can also close the milk container with added stopper and place in a refrigerator to store. After using, wash all parts following cleaning instructions. You can split the container in two parts and pour out the remaining breast milk for storage.

9. MILK STORAGE

- Due to the active biochemical and microbiological processes occurring in the milk, it is important to maintain a constant temperature of storage.
- Expressed milk store in bottles, special sterile bags or sealed containers with a lid (which should be disinfected before use).
- Mark each container with the date.
- Freeze only fresh milk. Do not fill a container fully as frozen breast milk increases its volume.
- Breast milk can be heated in the same way as any other food (but never in the microwave as this may destroy valuable nutrients and antibodies).
- Frozen breast milk can be defrosted overnight in the refrigerator and then warmed gradually in a bottle warmer or hot water.
- After defrosting, shake the container or stir the milk to mix the separated fat particles with the remaining volume of milk.
- DO NOT store milk on shelves placed on a refrigerator door.
- NEVER refreeze previously defrosted milk.
- DO NOT add fresh breast milk to defrosted milk.
- DO NOT add fresh breast milk to frozen milk.
- NEVER heat breast milk by boiling or putting it in the microwave.
- ALWAYS check the liquid temperature before feeding.
- Dispose of any milk that has not been used.

Type of breast milk	Temperature (°C)	Recommended time of storage
Fresh milk	+4°C	up to 96 hours
	from +4°C to +15°C	maximum 24 hours
	+19°C to +25°C (room temperature)	optimally up to 4 hours
Frozen milk	from +25°C to +37°C	maximum up to 4 hours
	-20°C	optimally 3-6 months
Thawed milk	+4°C	up to 24 hours (since total thaw)
	from +25°C to +37°C	up to 4 hours

10. TROUBLESHOOTING

Problem	Solution
Compatibility	Comes with 24mm flange , fits 24-22mm nipples. You can also buy 27mm flange insert fits nipples 27-25mm.
Little or no suction	Check whether the air chamber is clean and well, ensure that the front and back covers of air chamber and silicone diaphragm seal are flat and sealed. Make sure the silicone flange fits the breast perfectly. Are all accessories clean and dry.
If you press the Start/Stop button, the breast pump does not operate	Make sure the breast pump is charge. Make sure the breast pump is plugged in and/or charger is plugged.
If the outside of the breast pump gets wet	Disconnect , turn off and stop using. Allow the outside of the breast pump to dry.
If the breast pump is immersed in water	Unplug the breast pump from the socket and switch off the unit . Do not turn the breast pump upside down, the product must be facing up . Store the breast pump in a warm, dry place for 24 hours. Note: avoid direct sunlight and do not heat the breast pump to increase drying speed.
Small or no pump strength	Check that all connections to the breast pump and accessories are firm. Disassemble the double sides , breast pump accessories and check that all parts are clean and undamaged. Assemble the breast pump accessories as instructed in „ Assembling the Breast Pump „- Chapter 6). Ensure that the flange fits perfectly on the breast while pumping.
If there is breast milk spillage	Turn off the breast pump switch. Wipe the breast pump with a damp (not soggy) cloth. When all components are dry, reassemble.
Not „ lactating „ or not pumping milk	Make sure that the breast pump accessories are properly assembled and you can feel the suction. - If no milk comes out after two consecutive breast pumps, relax and rest for 10-15 minutes. - If no milk comes out , consult a doctor or breastfeeding specialist.

	Symptoms	possible causes
Breast swelling	Breasts become hard, feel uncomfortable, and some areas may become red.	The milk does not drain effectively from the breast. Milk can flow back into the tissues, causing swelling and pressure in certain areas.
Obstructed/Blocked milk ducts	Certain areas on the breast look red and may be very sensitive and painful when touched.	Milk ends up draining from some milk ducts. The corresponding area is blocked and the milk cannot flow.
Mastitis	It usually appears after breast swelling, but can occur suddenly. Initial symptoms are similar to the flu: fatigue, headache, muscle pain, fever and localized pressure in the breast. It may occur in a single breast or in both breasts at the same time and requires immediate treatment.	Bacterial infections in the breast tissue are usually accompanied by nipple rupture.

11. TECHNICAL PARAMETERS

1	Model	20/100
2	Battery	DC 3.7V/1000mAh
3	Operating voltage	DC 3.7V
4	Operating current	≤1000mA
5	Rate voltage	DC 5V 2A
6	Rate power	10W
7	Charging time	1.5±0.2H
8	Charging current	800±50mA
9	Lasting time	>1H
10	Negative pressure	≥60KPA
11	Vacuum value	≥60KPA
12	Flow rate	≥2.3L/min
13	Noise	≤50db(A)

14	Display	
15	Charging indicator light	Revolving LED light
16	Low voltage reminder	3.1V±0.2V, light flashing, shut down in 10S
17	Working indicator light	LED lights on
18	Mode	Mode A1 P1-P8 Mode A2 P1-P8 Mode A3 P1-P9
19	Working environment	Temperature -10 C ~+55 C, humidity93% RH, atmospheric pressure 700hPa~1060hPa
20	Storage environment	Temperature -10 C ~+55 C, humidity93% RH, atmospheric pressure 500hPa~1060hPa
21	Static Current	≤50uA
22	Screen goes off	No action in 20s

If you have any doubts about the safety of the product, please contact with distributor or manufacturer. The information contained in this manual is subject to appropriate technical changes.

This marking indicates that the product should not be disposed of with other household wastes. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of waste and recycle it resp. onsbly to promote the sustainable reuse of material resources. For further information on the recycling of this product, please contact the municipal authorities in their area of waste disposal service or the supplier of the product. This product and its information meet the requirements of the WEEE Directive 2012/19/EU.

The manufacturer declares that the product complies with European Union directives and harmonized standards. The product has been marked by conformity assessment procedures.

The product meets the requirements of the directive concerning electrical equipment designed for use within certain voltage limits for power adapter (LVD) 2014/35/EU, the Electromagnetic Compatibility Directive (EMC) 2014/30/EU, and RoHS Directive 2011/65/EU.

It is the Class III appliance

This product is not a toy. Keep all parts away from children. Keep the instructions of use as it contains important information. Batch code is placed on the packaging.

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DECLARATION OF CONFORMITY EU No. 01/24

Manufacturer: **Canpol Sp. z o.o.**
Pulawska Street 430, 02-884 Warsaw, Poland

Product name: **Wireless Electric In-Bra Breast Pump Hands-Free**

Brand: **Canpol Babies**

Model/ cat. no.: **20/100**

In relation to the described product, the manufacturer declare under the sole responsibility that the product to which this declaration refers is in line with the following European Union directives:

2014/30/EU Electromagnetic compatibility (EMC)

2011/65/EU Restrictions of the use of certain hazardous substances in electrical and electronic equipment (RoHS)

The product is in line with harmonized standards and other technical standards:

- EN 60335-1:2012 + AC:2014 + A1:2019 + A2:2019 + A11:2014 + A13:2017 + A14:2019 + A15:2021
- IEC 60335-1:2020 (not harmonized)
- EN 55014-1:2017 + A11:2020
- EN 55014-1:2021 (not harmonized)
- EN 55014-2:1997 + A1:2001 + A2:2008 + AC:1997
- EN 55014-2:2021 (not harmonized)
- EN IEC 63000:2018

The last two digits of the year in which the marking was applied

: 23

Warsaw, 05.01.2024

(Place and date of issuing the declaration)

(name, surname, position and signature of the authorized person)

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