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**KEDLEY**.co.uk**ORTHOPAEDIC SUPPORTS****ADVANCED FOAM ARM SLING**

[REF] KED056

**ENG ADVANCED FOAM ARM SLING**

Please read the following instructions and keep for future reference.

**KEDLEY Orthopaedic Foam Arm Sling** is designed to offer comfortable discreet, effective support and immobilization to the shoulder, arm and wrist. The weight of the forearm and wrist is carried by an adjustable foam strap that sits comfortably around the neck. Its contoured shape offers superior comfort and support.

The Kedley Orthopaedic range is made from only the highest quality materials. Each item is manufactured to give you steady support to promote healing and comfort.

**Note:** The support should be sized correctly to ensure maximum benefit and to avoid over compression. If the support is being used following an injury, seek medical advice if no improvement in the condition is seen within 72 hours. If the injury is severe or if in any doubt seek medical advice immediately. If used by children, adult supervision is required. Keep out of reach of young children. Always use on clean intact skin. Do not use if liniments, ointments or creams have been applied to the affected area.

**Warning:** If soreness or a rash occurs discontinue use immediately and seek medical advice.

*If any serious incidents occur in relation to using this device it should be reported to the manufacturer and the competent authority of the member state in which the user is established.*

**Washing & Storage Instructions:** Hand wash in lukewarm soapy water, rinse and air dry. Store in a cool, dry place and away from direct heat and sunlight.

**ORTHOPAEDIC SUPPORTS  
ADVANCED FOAM ARM SLING**

Website: [www.kedley.co.uk](http://www.kedley.co.uk)


**Impo International Ltd**  
Unit 3, 60 Lower Road,  
Harrow, Greater London,  
HA2 0DH, UK.


Eurolink Europe Compliance Ltd,  
Dublin, D02 AY86, Ireland.

REV: KED056IFU-05/21v1


- 1) Attach the Velcro end of the sling to the other end. Making a figure of 8 shape.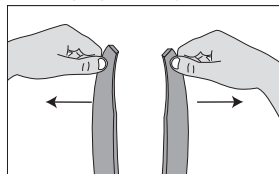


- 2) Slide hand through the smaller wrist loop and adjust so comfortable.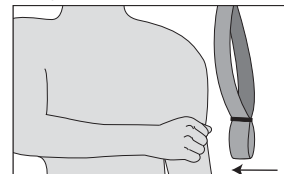

- 3) Put larger loop over your head and rest it behind your neck.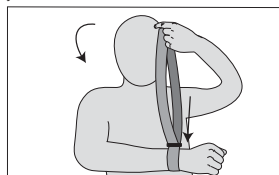

- 4) Adjust size so comfortable and arm is resting at the desired angle.

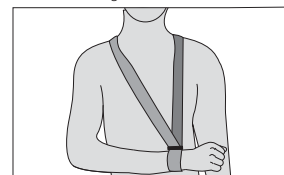**KEDLEY**

06052021

Video fitting instructions available @ [www.kedley.co.uk](http://www.kedley.co.uk)

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